

INSTRUCTIONS FOR THE CLEARPREP COLLECTION VIAL

1. Remove the cap from the ClearPrep vial.
2. After collecting the sample, place the portion of the device containing the cellular material into the preservative fluid in the vial. Rinse the device thoroughly to insure the release of the material, using the internal ridges of the vial to enhance this process.
3. Discard the collection device in the usual manner.
4. Replace and tighten the cap on the collection vial.
5. Assign the required patient information on the vial label and forward to the lab for processing.

NOTE: *It is strongly recommended that the collection device not be left in the vial.*

